

**DABIT-008**  
**(Opiates/Oxycodone/Heroin)**  
[www.fieldforensics.com](http://www.fieldforensics.com)



1) Remove **Sampler** from clear tube.

2) Remove paper tab from **Sticky Sampler**  
**Tip. DO NOT TOUCH** sticky sampler tip.



3) **Dab** sticky Sampler Tip on the suspect material.  
**Dab firmly** to collect a sample of suspect material.

4) **Return Sampler** to Clear Inner Tube, Sampler tip first, and close.



5) Hold Sampler as shown. **Squeeze Clear Inner Tube AND Sampler Tube** to break ampoule.



6) Keeping the **Sampler Tip** pointed downward, look for a color change on the **Sampler Tip** and compare to the color chart below. Observe for 60 seconds; if no color then the test is negative.

7) After recording results, carefully open clear tube and **SLOWLY** add five to six drops of the Neutralizer liquid (provided). Close cap on clear tube.

**Possible Irritant:**

**Avoid contact of liquids with eyes and mucous membranes.**

*December 2014 Rev. 8*

# FIELD FORENSICS INC.

## DABIT-008

### ***"Opiates Test" (Froede Reagent)***

Negative test

Heroin

Ecstasy  
(MDMA)



Presumptive test for Opiates, oxycodone, and heroin.

Full DABIT™ Color Chart Available at [www.fieldforensics.com](http://www.fieldforensics.com)